

# Bone Spreading

Bone Spreading is for placement of 2-3 of implants at narrow ridge




Cut the bone ridge using disk.  
Place spreading pins alongside of bone opening.  
Engage spreading adapter on spreading pin and turn.  
Spreading pin can be turned either with spreading adapter or latchet adapter.  
Turn 3 pins slowly to get even expansion.  
After spreading, remove spreading pin and place fixture.  
Spreading pin and fixture shall be replaced on by one, not all 3 pins at one time.  
Graft bone in between placed fixtures.

## Spreading Pin



## Disk

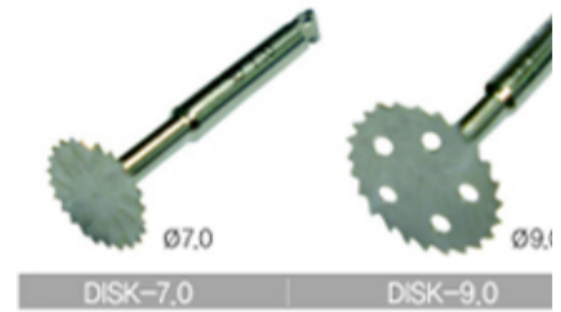

## Adapter



## Spreading Adapter



## Wrench



## Bone Spreading Kit

